

****Field Retrofit Required****

Request for immediate attention!

Dear Valued Customer,

Customers who have purchased Oxygen regulators before *Sept. of 2000*, should check to ensure that a filter has been placed into the high pressure inlet orifice. (see photo insert for clarification)

Regulators that do not have the bronze filter in place have the potential of a regulator fire. With the filter in place the risk of fire is greatly reduced, if not eliminated entirely.

Precision Medical, Inc. Regulators with the filter in place have passed the ignition testing requirement. ASTM G175 “Standard Test Method for Evaluating the Ignition Sensitivity and Fault Tolerance of Oxygen Regulators Used for Medical and Emergency Applications”

Please notify Precision Medical, Inc. by the following methods to indicate that you have received this letter and your appropriate response listed below.

Phone 800-272-7285 Ext 700
Fax : 610-262-9554
Email: regulator@precisionmedical.com
Web site: www.precisionmedical.info

1. You have received the notification, but have no affected regulators
2. You have received the notification and all regualtors have been retrofitted.
3. I have regulators that need to be retrofitted and I need _____ # of filters to complete the retrofit.
4. Please contact me because I have questions concerning this issue.

Customer Name _____

Address _____

Customer # _____

Phone # _____

Contact: _____

Sincerely,
Quality Assurance
Precision Medical, Inc.

Precision Medical Regulators



Easy DialReg



Easy GaugeReg



Easy MeterReg



No Filter



Filter